

Urilyzer[®] 100



The Urilyzer[®] 100 combines Analyticon's 30 years of experience in urine testing with an innovative instrumentation enabling a new way in urinalysis.

**A new way
in urinalysis**



In combination with the CombiScreen[®] 11 SYS PLUS urine test strips the system offers uncompromising ease of use, high quality and efficiency in a compact format.

The Urilyzer[®] 100 Pro additionally provides enhanced operator management and connectivity capabilities to meet the growing demands for compliance management and data capture.

The Urilyzer[®] 100 series is the most advanced urinalysis system available.

Start-Up Wizard

- On first usage the operator can select the language. Afterwards, the Start-Up Wizard leads through the user-defined settings
- The Start-Up Wizard enables the user to perform measurements without having to go through the complete User Manual



Autostart function

- Automatic start of the measurement after placing the urine test strip on the strip holder
- Simplifies operation of the analyzer and makes the workflow more independent of the user
- Hygienic and clean operation of the analyzer



Convenient connectivity

- Data can be transferred via serial connection or Ethernet (Pro version only)
- A variety of interfaces for connecting external barcode scanner and/or keyboard (USB or PS2)
- Software updates are possible anytime via USB-stick

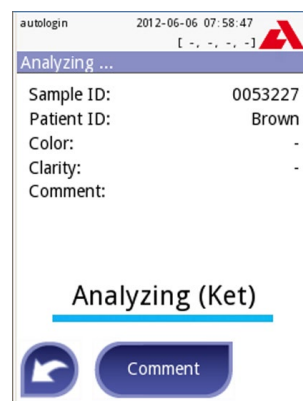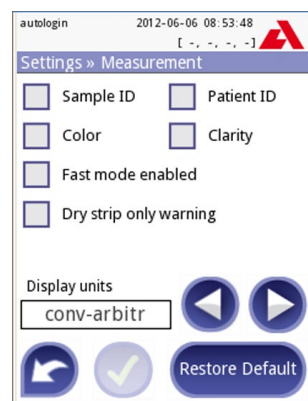

Color touch screen

- Intuitive icons on a high-contrast, color touch screen navigate the user through the menu
- Positive results, reminders and warnings are shown in color (e.g. red or yellow) and can be easily identified
- The flexible menu offers a lot of automated functions and is customizable to the specific user needs

The Urilyzer 100 offers a broad range of optimized usability features.

The intuitive user interface improves data entry and allows manual entry of color and turbidity from a predefined list or by an additional comment. Moreover, the system provides the user with a high level of customization with flexible testing and reporting options. The intelligent data management provides multiple filter options. The Pro version additionally offers an automated QC analysis with QC test reminders and supports a lot code documentation. In terms of system security the Pro version allows the allocation of different security levels to individual users.



Improved security

- Tracking of LOT-No. for urine strips and quality control solutions
- After loading the strip into the strip holder, the system checks the correct positioning



Quality Control Management (Pro version)

- Customizing of the QC protocol according to regulatory requirements and laboratory standards
- Enables QC reminding functions or lockout protocol in a customizable time interval
- QC results are compared to the entered reference values and the last 1000 QC records are stored

Operator Management (Pro version)

- Unauthorized use is prevented by using either a predefined security level or an individual customised security level
- Measurements and Quality Control are tracked to a specific operator ID (Password protected)



Quiet and fast printer

- Printer with Easy-load function makes it very easy to change the printer paper. Just insert the paper and start printing
- The integrated printer is fast and quiet

Urilyzer® 100 – A new way in urinalysis

	Urilyzer® 100	Urilyzer® 100 Pro
Intelligent data management	■	■
Autostart	■	■
Advanced data entry (e.g. color, clarity, lot, expiry)	■	■
Automatic print-out	■	■
Customized testing and reporting options	■	■
Serial interface to host computer / LIS	■	■
Optional barcode scanning	■	■
Recommendation for additional sediment evaluation	–	■
Memory capacity		
• test results	up to 1000 patient tests	up to 3000 patient tests
• QC testing	up to 500 results	up to 1000 results
Operator management	–	■
Adaption of the system security	–	■
Ethernet connectivity	–	■
HL7 protocol	–	■

Technical specifications

Type:	reflectance photometer wavelengths: 505, 530, 620, 660 nm
Throughput:	up to 120 tests/hour in fast mode and up to 50 tests/hour in normal mode
Memory:	patient tests: 1000 / 3000 (Pro version) and QC tests: 500 / 1000 (Pro version)
Display:	3.5" QVGA touch-screen LCD
Interfaces:	RS232 serial port, USB A, USB B, PS2 (external keyboard, barcode reader), microSD card, Ethernet port (Pro version)
Dimensions/Weight:	190 x 260 x 77 mm (WxDxH) / 1.5 kg
Electrical rating:	100–240 V AC, 50/60 Hz



Urine Strip Analyzer
Urilyzer® 100
Urilyzer® 100 Pro

Art.-No.
UL0100
UL0100Pro

Urine Test Strips
CombiScreen® 11SYS PLUS
CombiScreen® 7SYS PLUS
CombiScreen® 5SYS PLUS

Pack size **Art.-No.**
100 Streifen 94100
100 Streifen 94110
100 Streifen 94109

Not available for sale in the U.S.

Analyticon Biotechnologies AG

Am Muehlenberg 10
35104 Lichtenfels - Germany

Fon +49 (0) 6454 7991-0
Fax +49 (0) 6454 7991-71

info@analyticon-diagnostics.com
www.analyticon-diagnostics.com